


# Bischoff & Bischoff


Instruction manual

**BB-80**



Enjoy mobility.

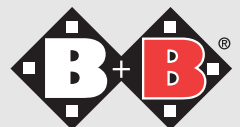

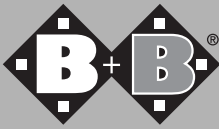

**Dear customer,**  
with the purchase of this B+B bath board you are now the owner of a high-quality German product, thousands of which are already in professional use. This instruction manual will familiarise you with the use and maintenance of your B + B bath seat and will assist in making your life safer. We hope you enjoy our product.

Showerhead holders

Handle  
Soap dish

## **1. Attention - safety instructions! (Risk analysis as per DIN EN ISO 13485)**


- Max. load capacity 100 kg
- Use the suspended seat only in the temperature range of +3°-+70°.
- Do not sit on the front edge of the bath board, danger of falling



Please follow the installation instructions on pages 2 and 3, and the safety instructions



Please read the instructions for use and installation carefully. Take special note of the safety instructions.

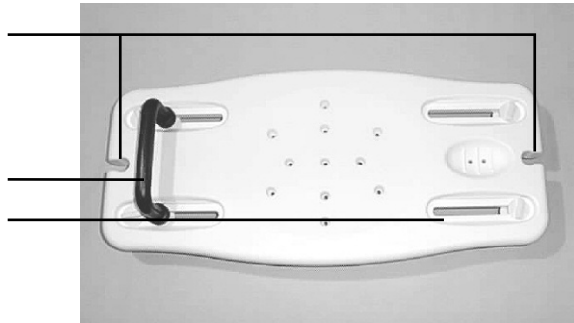

The BB-80 bath board will allow you to remain comfortably seated in the bath. The integrated soap dish and the showerhead holders on both sides of the bath board are also very practical aids.

## 2. View from below

### Antislip pad

The antislip pads prevent the bath board from slipping and protect the edge of the bath tub from damage.

### Antislip pad socket

Press the pads straight into the sockets.

## 3. Distance between the feet

Before tightening them adjust the distance of the feet to match the distance between the sides of the bath. The distance can be set accurately for all standard bath tubs. Then tighten the screws.

Min. distance: A = 45 cm

max. distance: B = 72 cm

W/H/D: 79 / 14 / 37 cm

Weight: 2,4 kg



Fig. 1


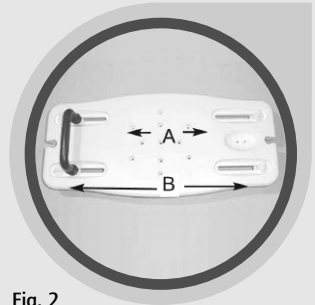


Fig. 2

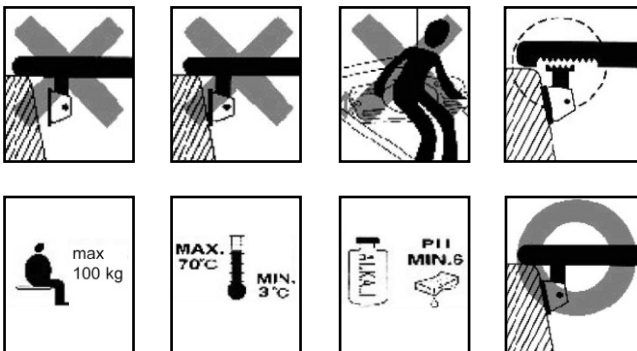

## Warranty section

(Please note our general terms and conditions)

**Model:** BB-80 bath board

**Serial number:\***

(to be entered by dealer)

**Dealer:**

**Date and stamp**

\*See the side frame for information stickers

## Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 72 48/92 09-2 • Fax: +49 72 48/92 09-3

[www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

[international@bischoff-bischoff.com](mailto:international@bischoff-bischoff.com)

