


# Bischoff & Bischoff


Instruction manual

## Curacao



Enjoy mobility.

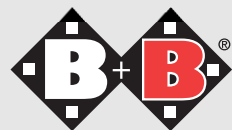

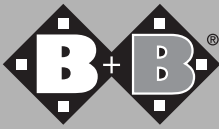

## **Curacao hanging seat (without seat back)**

**Dear customer,**

with the purchase of this B+B shower stool you are now the owner of a high-quality german product that will make home care much easier.

This instruction manual will familiarise you with the use and maintenance of your B+B shower stool and make your life safer.

We hope you enjoy your new product.

### **1. Declaration of Conformity**

B+B as the manufacturer of the product declares that the Curacao hanging seat fully conforms to the requirements of Directive 93/42/EEC.

### **2. Information on using the product**

The Curacao hanging seat is designed exclusively for use while seated in a bath tub for washing.

### **3. Purpose - indications**

Significantly reduced function and lack of function of the lower extremities (e.g. amputations, paralysis, joint diseases, neuromuscular diseases), consequences of chronic disease with reduced strength in the lower and upper extremities.

### **4. Caution - safety instructions - Risk analysis as per DIN EN ISO 13485**

- An assistant is required for entering and leaving the seat
- Max. load capacity 120 kg
- When the seat is permanently fastened to the frame beware of the danger of becoming jammed.
- Seat is designed for sitting only
- Make sure that the side sections are securely fastened to the inside of the bath tub and cannot slip.

### **5. Use**

The Curacao hanging seat is suspended in the bath tub so the antislip feet are on the edges of the bath tub.

## 6. Assembly and installation

Insert sides into the seat frame and fasten from below with the threaded pins (see Fig. 1 and 2).

Seat frame

Side frame

Seat

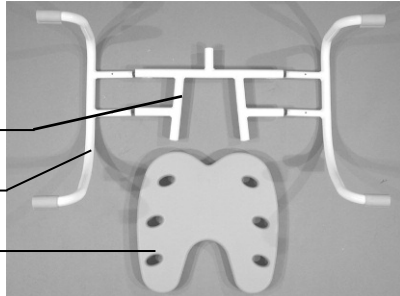


Fig. 1

### Attention

When assembling the frame please note:

- The part of the seat frame with the 90° bend must point to the back of the seat
- The opening in the seat is to the front

## 7. Adjusting the width of the seat

To adjust the hanging seat loosen the threaded pins under the seat frame and pull the side sections out (see Fig. 2)

Caution!

The red dots mark the maximum width of the hanging seat (see Fig. 3).

## 8. Installing and removing the seat

The seat is locked to the seat frame by pressing it down forcefully (see Fig. 4).

To remove the seat hold it by the side of the frame and pull the seat from the frame (see Fig. 5).

## 9. Cleaning and disinfecting the hanging seats

Clean and disinfect the seat with standard cleaners and disinfectants only.

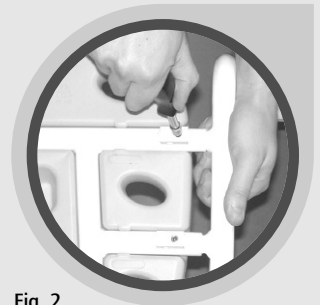


Fig. 2

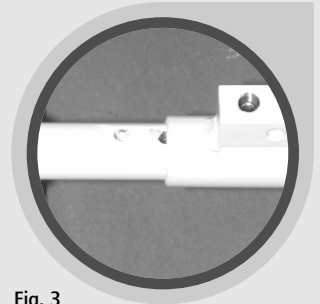


Fig. 3



Fig. 4


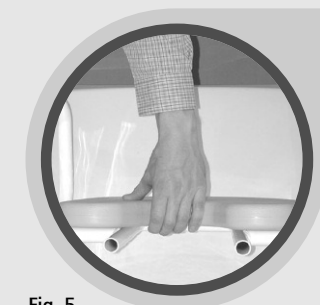


Fig. 5

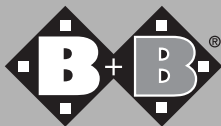

## **10. Technical data**

- Max. load capacity: 120 kg
- W/H/D (in mm): 690-790 / 520/ 270
- Weight: 3.1 kg.
- Seat width/depth: 400/400 mm
- Adjustment range: 590 to 690 mm (inside of bath tub)
- Heat resistance: up to 125°C
- Colour: blue or white

## **11. Adjustment and repair tools**

- 1 x Philips screwdriver
- 1 x Allen key no. 5

## **12. Marking of the product**

An identification sticker is attached to the frame of the product with the product identification/ serial number/ max. user weight/ company address.

## **13. Warranty**

A warranty claim can only be accepted if the product is used under the specified conditions and for the specified purposes.

The product must be returned in the original packaging.

## **14. Standard equipment**

- 1/1 x side section left/right
- 1 x B+B seat
- 1 x seat frame

## **Warranty certificate**

(Please note our general business terms and conditions)

**Model:** Curacao hanging seat

**Serial number:\***

(Please note our general business terms and conditions)

**Dealer:**

**Date and stamp**

\* See the side frame for information stickers

## **Bischoff & Bischoff GmbH**

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 700/6000 7070 • Fax: +49 700/6000 8080

[www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

[info@bischoff-bischoff.de](mailto:info@bischoff-bischoff.de)

