


Bischoff & Bischoff


Instruction manual

DH-40 Round



Enjoy mobility.

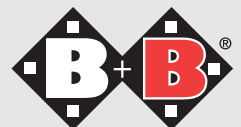

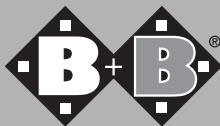

DH-40 Round (shower stool)

Dear customer,

with the purchase of this B+B shower stool you are now the owner of a high-quality German product that will make home care much easier.

This instruction manual will familiarise you with the use and maintenance of your B+B shower stool and make your life safer.

We hope you enjoy your new product.

1. Declaration of Conformity

B+B as the manufacturer of the product declares that the DH-40 round shower stool fully complies with Directive - 40 93/42/EEC.

2. Information on using the product

The DH-40 round shower stool is designed exclusively for use while showering.

3. Purpose - indications

Poor balance and inability to stand with reduced use of lower extremities (e.g. loss of strength with malignant diseases, chronic, advanced cardiovascular disease and lung diseases) combined with weakness in moving.

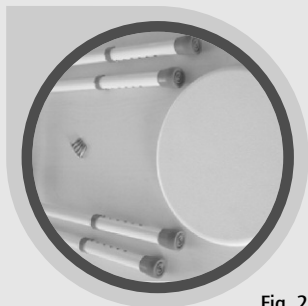


Fig. 2

4. Caution -safety instructions - Risk analyses as per DIN EN ISO 13485

- An assistant is required for entering and leaving the seat
- Set all four feet to the same height.
- Make sure that the locking pins are correctly seated
- When the seat is permanently fastened to the frame beware of the danger of becoming jammed.
- Seat is designed for sitting only

5. Assembly and installation

Insert telescopic tubes into the slots in the seat as shown in the picture and fasten with the five cylinder bolts.

Telescopic tubes

Seat

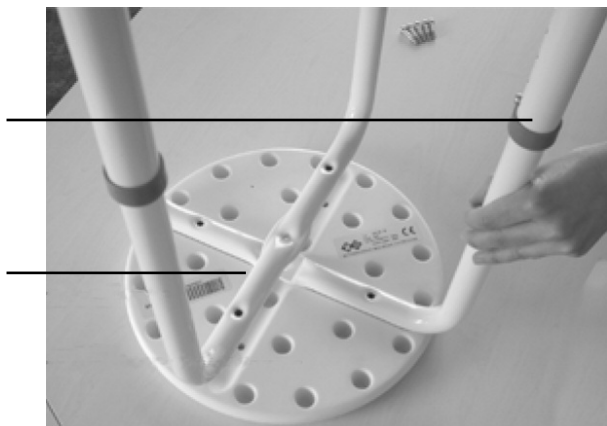


Fig. 3

6. Adjusting the height of the seat


The two feet are height-adjustable in six positions (adjustment range 10 cm). Press the locking pin and slide the telescopic foot to the desired position. The locking pin automatically locks when it comes to the hole. All four feet must be set to the same height.

7. Cleaning and disinfecting the shower stool

Clean and disinfect the seat with standard cleaners and disinfectants only.



Fig. 4

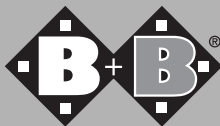

8. Technical data

- Max. load capacity: 120 kg
- W/H/D (in mm): 420 / 420 / 440-540
- Weight: 1.7 kg
- Seat height: 440-540 mm 6 positions
with locking pin
- Seat width/depth: 330 / 330 mm
- Colour: white

9. Marking of the product

An identification sticker is attached to the frame of the product with the product identification/serial number/max. user weight/ company address.

10. Warranty

A warranty claim can only be accepted if the product is used under the specified conditions and for the specified purposes. The product must be returned in the original packaging.

11. Standard equipment

- 1 x B+B seat, plastic
- 2 x telescopic legs with height-adjustable feet
- 5 x M6 x 28 cylinder bolts

Warranty certificate

(Please note our general business terms and conditions)

Model: DH-40 Round

Serial number:*

(Please note our general business terms and conditions)

Dealer:

Date and stamp

* See the side frame for information stickers

Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 700/6000 7070 • Fax: +49 700/6000 8080

www.bischoff-bischoff.com

info@bischoff-bischoff.de

