


# Bischoff & Bischoff


Instruction manual

**TSE-A / O**



Enjoy mobility.

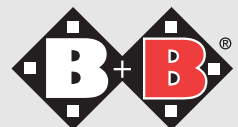

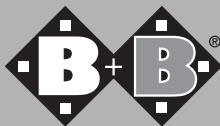

## **TSE-A/O**

**(Toilet seat raiser)**

**Dear customer,**


with the purchase of this B+B toilet aid you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B + B toilet aid and make your life safer.

We hope you enjoy our product.

## **1. Attention safety instructions! (Risk analysis as per DIN EN 13485)**

- Max. load capacity 150 kg
- The load capacity is not based on the cover of the commode seat. The cover is not suitable for use as a seat.
- The toilet seat raiser is not suitable for continuous use as a seat.
- Make sure that the arm rests are fully locked in position.
- When unfolding the arm rests make sure that you do not jam parts of the body or clothing.
- Make sure that the fixation screws are tight.
- Set the front and back height adjustment to the same height.

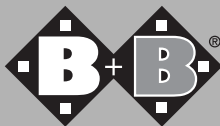

## **2. Installing the arm rests (TSE-A only)**

Insert the arm rests into the hole in the rear housing section of the toilet seat raiser. Press the spring button with your thumb and hold it down until the spring button is inserted through the hole.

Before lowering the arm rests make sure that no parts of your body or clothing are jammed. It is also important to note that the arm rests are fully locked before supporting it.

## **3. Adjusting the height of the seat**

The toilet seat raiser can be set to three heights (8, 12 and 16 cm). To adjust the height of the seat unscrew the two knurled screws at the back of the TSE.

- Remove the plastic caps in the required position.
- Lift the aluminium rod into the desired position and tighten the knurled screws again.
- Press the remaining plastic caps into the open holes.
- Proceed as described above at the front of the TSE.

Make sure that the back and front aluminium rods are set to the same height.

#### 4. Installing the toilet seat raiser

- Unscrew the cover of the toilet.
- Screw the two long plastic nuts to the back aluminium rod. Insert the two threaded rods into the holes in the toilet.
- Screw the two long plastic nuts to the threaded rods from below.

Depending on the size of the toilet, it is necessary to mount the rear aluminium support rod rotated with the fixation screws rotated 180° to prevent the front support rod from slipping of the toilet bowl (see photos to the right).

Make sure that the nuts are tight to hold the TSE-Easy firmly on the toilet



Minimum distance between toilet  
and wall 18 cm.

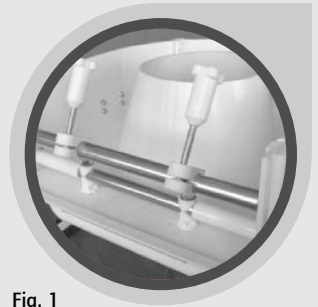


Fig. 1


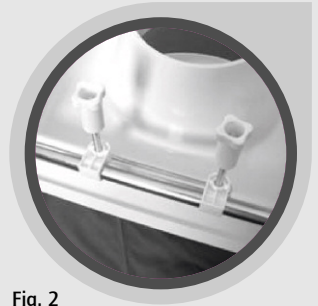


Fig. 2

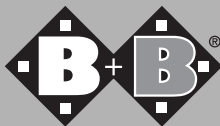

## **5. Arm rests (TSE-A only)**

The arm rests can be folded back. The arm rests can be locked in the down position by lightly pressing down on them (click noise).

The arm rest must be folded back for transfer from a wheelchair. Follow instructions in section 1 (installing the arm rests).

## **6. Care and maintenance**

Clean and disinfect the toilet seat raiser with standard cleaners and disinfectants. Do not use aggressive cleaning agents. The toilet seat raiser is maintenance-free.

## **7. Technical data**

Depth:	49 cm
Width:	58 cm
Height:	37 cm
Arm rest height (from toilet seat):	21 cm
Seat width:	40 cm
Seat depth:	40 cm
Weight of TSE-A:	6,3 kg
Weight of TSE-O:	4,4 kg
Max. user weight	150 kg

## **Warranty certificate**

(Please note our general terms and conditions)

**Model:** TSE-A/O toilet seat raiser

**serial number:\***

(to be entered by dealer)

**Dealer:**

**Date and stamp**

\* See the side frame for information stickers

## **Bischoff & Bischoff GmbH**

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 (0)700/6000 7070 • Fax: +49 (0)700/6000 8080

[www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

[info@bischoff-bischoff.de](mailto:info@bischoff-bischoff.de)

