

# Bischoff & Bischoff

Instruction manual

## Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 (0)700/6000 7070 • Fax: +49 (0)700/6000 8080


[www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

[info@bischoff-bischoff.de](mailto:info@bischoff-bischoff.de)

## TS-200



Enjoy mobility.

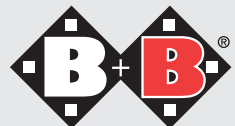

## TS-200 (commode seat)

### Dear customer,

with the purchase of this B+B commode seat you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B+B commode seat and make your life safer.

We hope you enjoy our product and are more mobile with it.

### 1. Attention - safety instructions! (Risk analysis as per DIN EN 13485)

- Max. user weight 120 kg!
- The commode seat should only be used indoors and with the aid of an assistant.
- Push the seat on even and horizontal surfaces only.
- Always lock both brakes when at a standstill.
- Do not lean out to the side. Danger of tipping!
- Make sure that the seat-back screws are tight.
- The commode seat must not be used by unauthorised persons (particularly children).

Commode wheelchairs are recommended if the patient cannot use or access toilette because of the disability.

- Back cover
- Adjustable arm rest with padding
- Locking pin for arm rest
- Seat cushion
- Fixation screw for back rest
- Locking pin for leg support
- Commode with cover
- Leg supports
- Back steering wheel, braked
- Front steering wheel, unbraked



Fig. 1

## 2. Parking brake

The two rear steering wheels with brakes prevent the commode seat from moving accidentally.

- Locking: press brake lever 1 down to the limit stop with the foot (Fig. 2)
- Releasing: press lever 2 to release the brake (Fig. 2).



Fig. 2

## 3. Leg supports

To make it as easy as possible to enter and leave your B+B commode seat, the leg supports can simply be swung to the side out of the way. It is not necessary to remove them.

- Pull the locking pin slightly out (Fig. 3) to open the lock.
- Swing the leg supports outwards.
- If necessary, fold the foot plates up first.



Fig. 3

To restore the original status, just swing the leg supports forward and lock them in place. To remove the leg supports proceed as follows:

- Pull the locking pin out (Fig. 3) to open the lock.
- Swing the leg support outwards.
- Pull the leg support out upwards.



### Danger of accident

Be careful not to stand on the foot plates and/or the foot supports when getting out of the commode seat. Danger of accident!



Fig. 4

#### 4. Commode seat

Removing commode seat:

- Pull front edge of seat up from the base.
- Lift commode seat upwards.

Positioning commode seat:

- Slide back edge of seat with the open corners to the back.
- Press front rounded edge of seat down to the stop. Seat must lock into position.
- Make sure that it is locked.

#### 5. Arm rests

The arm rests can be folded back.

- To fold them back pull out the locking button and fold the arm rest back (Fig. 4).
- To return it to its original position fold the arm rest forward to its original position until you hear the locking pins click into place.



Fig. 5



Make sure that the arm rests are locked in position.

#### 8. Technical data

Seat width:	45 cm
Seat depth:	44 cm
Seat to arm rests:	24,5 cm
Seat back rest:	33 cm
Depth:	94,5 cm
Height:	97,5 cm
Width:	53,5 cm
Max. user weight:	120 kg
Weight:	15,4 kg

### Warranty section

(Please note our general terms and conditions)

**Model:** commode seat TS-200

**Model serial number:\***  
(to be entered by dealer)

**Dealer:**

Date and stamp

\* See the side frame for information stickers