Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

RL-Smart


DE

EN

ES

FR

Enjoy mobility.

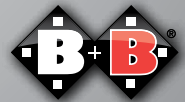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Index

1.	Introduction	21
1.1.	Signs and symbols	21
2.	Important safety Instructions	22
2.1.	General safety Instructions	22
2.2.	Indications / Contra-indications	22
2.3.	Purpose	23
2.4.	Declaration of Conformity	23
2.5.	Responsibility	23
2.6.	Product life	23
3.	Description of product and contents	24
3.1.	Inspection on delivery	24
3.2.	Type plate and Serial number	25
3.3.	Contents	26
3.4.	Summary	26
4.	Assembly/adjustments	27
4.1.	Unfolding	27
4.2.	Folding up	27
4.3.	Installing the functional bag	27
4.4.	Adjusting the handles	27
4.5.	Brake application and adjustment	28
4.7.	Wheels and tyres	29
5.	Use	29
6.	Technical data	30
7.	Maintenance tips for users	31
7.1.	Cleaning and disinfection	31
7.2.	Rollator check by the user	31
7.2.1.	Brake Inspection	31
7.2.2.	Checking the tyres	32
8.	Maintenance tips for retailers	32
9.	Storage/shipping	33
10.	Re-use	33
11.	Repairs	33
12.	Disposal	34
13.	Warranty certificate	35

1. Introduction

Dear Customer,

Congratulations on your decision to purchase this rollator. It is a German product that will improve your everyday life and bring you much enjoyment.

Before you take your first steps with the rollator, we ask that you read this instruction manual carefully and follow our recommendations. This will help you get to know your walking aid and ensure maximum safety when using it day-to-day.

As you will find out, the rollator is very simple to use. We are sure that you will get used to it very quickly and soon won't be able to imagine life without it.

Take full advantage of your new-found mobility!

With kind regards
You B+B team

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

2. Important safety instructions

2.1. General Safety instructions

- Max. capacity Rollator RL-smart 136 kg
- Please note there is a danger of trapping fingers and/or clothing when folding and unfolding. Do not catch any clothing or body parts between the moving parts.
- Please note the minimum insertion depth of the handles bars!
- Each brake only affects one running wheel!
- The bag can be loaded with max. 5 kg.
- When cleaning, do not use any aggressive or corrosive cleaning agents!
- Check the rollator for correct assembly before every use. The brake operation should be checked.
- Make sure your brakes are operating properly before each use.
- The surfaces of the rollator can heat up when in direct exposure with the sun. This may result in injury.
- Reporting incidents. The operator or user must immediately report
 1. any malfunction
 2. any change to the features or performance as well as
 3. any inaccuracies in the marking or operating instructions of a medical device, which has or could have led to death or to a serious deterioration of the health of a patient, an employee or a third party, to the Federal Institute for Drugs and Medical Devices (BfArM). This institute will immediately forward the notification to the operator's competent authority and also inform the manufacturer and the manufacturer's competent authority.

2.2. Indications / Contra-indications

- The rollator supports users with limited mobility to move, e.g. for domestic use, when shopping or walking outdoors
- The seat area allows a place to rest during longer distances.
- The rollator must be used on level and solid ground only.
- The product must not be used as a climbing aid or means of transport for heavy loads.

2.3. Purpose

The rollator is intended to support mobility of persons with an impaired ability to walk, in accordance with the above-mentioned indications.

We are only able to issue a guarantee if the product is used in accordance with the specified conditions and for the specified purpose.

2.4. Declaration of Conformity

B+B, as manufacturer of the product, declares that the rollator fully meets the requirements of the guideline 93/42/EEC.

2.5. Responsibility

We are only able to provide a warranty if the product is used in accordance with the specified conditions and for the intended purposes. If modifications, repairs or maintenance works are made, please make sure only authorised persons perform these works in accordance to the instructions of B+B.

2.6. Product life

The expected product life of the rollator amounts up to 5 years if used as intended and in compliance with the safety, care and maintenance instructions. Over this period and beyond, the rollator can be used if the product is in a safe condition.

3. Description of product and contents

3.1. Inspection on delivery

All B+B rollators are subject to a proper final inspection in our house and receive the CE mark.

The rollator is supplied in a special box. After unpacking, please save the box if possible. You may need it later for storage or to return the product.

Assembly, adjustment to your body size and first instructions are taken care of by your qualified mobility dealer.

If delivered by rail or a forwarding agent, the product is to be checked with regard to transportation damage at once in the presence of the shipper.


Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service (Tel.: +49 700 6000 7070).

We are only able to issue a guarantee if the product is used in accordance with the specified conditions and for the specified purpose.

3.2. Type plate and Serial number

The type plate (Fig. 1) and serial number are located on the bottom of the frame.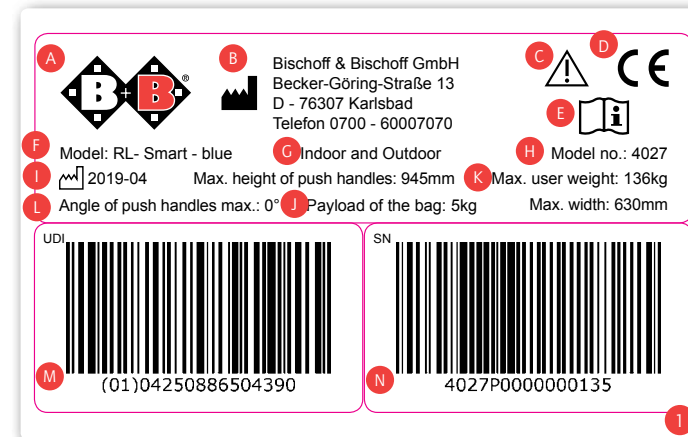

- A Manufacturer's logo
- B Manufacturer's information
- C Caution! - Follow instructions for use
- D CE mark
- E Important! - Follow instructions for use
- F Model designation
- G Application class
- H Model number
- I Date of manufacture
- J Carrying capacity of the bag
- K Maximum load capacity
- L Required dimensions
- M UDI number
- N Serial number

3.3. Contents

After receiving the goods, please check the contents for completeness right away. The content consists of:

- Outer package
- Rollator pre-assembled
- Instruction manual

3.4. Summary

Product overview RL-Smart (Fig. 2):

2

4. Assembly/adjustments

4.1. Unfolding

- Position the B+B Rollator in front of you in the direction of travel.
- Apply the parking brakes.
- To unfold the rollator, push the frame of the sit downwards (Fig. 3). Make sure that the frame is laid out correctly and that it clicks into place.

4.2. Folding up

- Apply the parking brakes.
- Hold the rollator firmly with one hand on the handle. With the other hand, pull the strap on the seat upwards (Fig. 4).
- You need to push past the resistance of the locking mechanism.

4.3. Installing the functional bag

- The B+B Rollator is unfolded. Apply the parking brakes.
- Position yourself with your back towards the direction of travel in front of the B+B Rollator.
- Pull the straps of the bag over the handle and slide these under the pins (Fig. 5).

4.4. Adjusting the handles

- The B+B Rollator is unfolded. Apply the parking brakes.
- Loosen the nut (Fig. 6) and remove the fixation screw.
- Stand behind the B+B Rollator and allow your arms to hang loosely down.
- Now insert the handles so they are at the height of your wrists.

- Secure the screw and tighten the nut.
- Repeat the process for the other side. Make sure that both handles are at the same height.

4.5. Brake application and adjustment

Service brake

The service brake is operated by both hand brake levers on the handles by pulling upwards (Fig. 7). Service and parking brakes affect the running wheels.

Parking brake


- To lock, push both brake levers downwards (Fig. 8).
- To release, pull both brake levers back to the central position.

Brake adjustment

The brake is set properly, if the brake pins do not touch the wheel and the locking brake can be activated easily.

- To adjust the brake to make it harder, unscrew the knurled-screw (Fig.8) on the braking mechanism and lock it with the other nut.
- With the adjusting of the Bowden cable (M6), loosen the locknut first.
- Next the threaded sleeve is adjusted until the correct adjustment is found. To prevent the setting from changing, the threaded sleeve (M6) must be locked with the nut (M6).
- To adjust the brake to make it softer, follow the above procedure in reverse.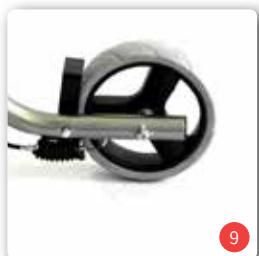

4.6. Wheels and tyres

Your rollator is fitted with puncture resistant PU wheels with sizes 190 x 43 mm at the front and 170 x 43 mm at the back.

5. Use

Walking

Make your first attempts to walk with the rollator on a flat, firm surface. Familiarise yourself with the function of the brakes. Let someone accompany you during this initial period.

For an anatomically correct gait, the push handles must be set at proper height (see sec. 4.4).

- Place yourself in the rollator so that your ankles are at the height of the rear wheels.
- Stand with good posture.
- Grasp the push handles with both hands and your arms bent at a slight angle.

When walking with the rollator try to stay as close as possible with the body to the push handles. This increases the stability and safety against tipping over; in addition, avoid a crooked back. Try to generally walk in the rollator and not behind it. Take small steps and do not push the rollator too far ahead.

You can determine the change of direction and speed by pushing the rollator. Always use the brakes to slow down your movement. The rollator could otherwise turn with the unbraked wheel and the braked wheel, and could fall over.

Go around obstacles whenever possible. Try to use ramps or take lifts. If this is not possible, the rollator must be carried over the obstacle. Find a second person to help you do this, if possible

6. Technical data

Type:	Lightweight rollator
Colour:	4026 platin / 4027 metallic blue
Height:	790 - 920 mm
Length:	740 mm
Width:	630 mm
Width between the handles:	490 mm
Width of the handles:	30 mm
Seat height:	540 mm
Seat width:	450 mm
Folded:	240 mm
Turning circle:	865 mm
Weight:	6,72 kg
Max. capacity:	136 kg
Max. load of bag:	5 kg

- Complete with walking stick holder and bag.
- Anatomically shaped handles.
- Integrated service and parking brakes.
- Stable aluminium construction, corrosion-proof coating.
- Safe turning handles, adjustable to 830 - 960 mm.

7. Maintenance tips for users

7.1. Cleaning and disinfection

- The frame and the paneling of the wheelchair can be cleaned with a moist cloth. You can use a mild detergent for more stubborn dirt.
- The wheels can be cleaned with a wet synthetic fiber brush (do not use a wire brush!)
- Push handles, arm pads and seat pads can be washed off. Use a mild detergent.
- Use a disinfectant listed by VAH for disinfection..

7.2. Rollator check by the user

7.2.1. Brake Inspection

Make sure your brakes are operating properly before each use.

It should not be possible to push the rollator away while the brakes are activated.

The rollator should be easy to drive without grinding noises and with good direction stability when the brakes are released.

If you notice unusual behaviour when braking, inform your qualified mobility dealer immediately and do not use the rollator.

7.2.2. Checking the tyres

Check whether the tyres are intact every time before driving.
If the tyre is damaged, it must be repaired by an authorised dealer.

8. Maintenance tips for retailers

If faults or defects at the rollator are found, it must be returned to the dealer for repair immediately.

The dealer can receive the required information and documentation for repair and service on request. A training program is also available for dealers.

B+B customer service is available at +49 (0)700/60007070.

The rollator must be inspected in accordance to the service plan and must be in good condition.

9. Storage/shipping

In the event that the rollator will be stored or shipped, it is advised to keep the original packaging to store in it for this purpose, so that it is on hand if necessary. Then your rollator can be optimally protected against environmental influences during storage or transport.

10. Re-use

The rollator is suitable for re-use. For re-use, the rollator should firstly be thoroughly cleaned and disinfected in accordance with the enclosed care and cleaning instructions. Finally, the product should be professionally checked for general condition, wear and tear and damage and if necessary, repaired by an authorised specialist..

11. Repairs

Repairs and maintenance measures are only permitted to be carried out by authorised partners (medical houses or specialist suppliers).

For a partner near you, please call our customer service team:
Tel.: +49(0)700 / 6000 7070.

12. Disposal

A worn out rollator has a valuable source of material, which can be recycled into the industrial cycle. Therefore, do not throw your rollator away with household waste, but allow it to be professionally disposed of by a state approved disposal department. Your local rubbish collection will have information regarding this. The environment will thank you.

If you wish to dispose the rollator yourself, please contact a local recycling company for the disposal regulations of your local jurisdiction.

13. Warranty certificate

Guarantees refer to all faults of the product that are attributed to material or manufacturing faults. In the event of complaints, the complete, filled in warranty certificate shall be submitted.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of B+B will generally lead to forfeiture of warranty and product liability claims.

Our products come with a statutory guarantee of 2 years. For replacement devices and repairs, a warranty period of 1 year applies.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

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Bischoff & Bischoff GmbH
B+B France S.A.R.L.